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List of cleaning and disinfection products for Stamskin and Batyline -Validity 15.2.2013

Product	Disinfectant	Producer	Use	MLI	Results	Recommendation	Date
Bacillol 30 Foam	Yes	Bode Chemie	Pure	180a	Light hardening with use time	Ok, rinse it properly	19.04.12
				180b	Yellowing with time	Nok	
Kohrsolin FF	Yes	Bode Chemie	Pure	181a	Hardening with time	+/-, bien rincer	
				181b	Light hardening with use time	Ok, rinse it properly	
Incidin Extra N	Yes	Ecolab	0.5%	182a	Tracks on surface, no hardening	Ok, rinse it properly	03.12.12
ClearSurf	Yes	Fresenius Medical Care	1%	183a	Tracks on surface, no hardening	Ok, rinse it properly	03.12.12
Frekanol N	Yes	Fresenius Medical Care	Pure	184a	Tracks on surface, no hardening	Ok, rinse it properly	03.12.12
Surfa'Safe	Yes	Laboratoires Anios	Pure	185a	Ok, light fatty surface with drops	Ok	02.07.12
				185b	Ok	Ok	
WIP Anios Premium	Yes	Laboratoires Anios	Pure	186a	Ok	Ok	13.09.12
Nettoyant Désinfectant Multisurface	Yes	Laboratoires Anios	Pure	187a	Ok, light tracks on surface	Ok	13.09.12
X-Spray	No	Laboratoires Anios	Pure	188a	Hardening	Nok	13.09.12
All Clean	Yes	Joli	Pure	189a	Ok	Ok	03.03.12
				189b	Ok	Ok	
Antifect Extra	Yes	Schülke +	Pure	190a	Ok	Ok	11.01.13
				190b	Ok	Ok	
Terralin Protect	Yes	Schülke +	Pure	191a	Tracks on surface, hardening	Nok	11.01.13
				191b	Light hardening	Ok	
Incidin Rapid	Yes	Ecolab	5%	192 a	Tracks on surface, light hardening	Nok	15.02.13
				192 b	Yellowing	Nok	
Optisal N	Yes	Dr Schumacher	5%	193 a	Tracks on surface	Ok	15.02.13
				193 b	Strong yellowing	Nok	
Detergent Désinfectant Surfaces Hautes (DDSH)	Yes	Laboratoires Anios	Pure	194 a	Light tracks on surface	Ok	15.02.13
				194 b	Yellowing	Nok	
Mousse Detergente des surfaces	Yes	Laboratoires Anios	Pure	195 a	Light tracks on surface	Ok	15.02.13
				195 b	Light Yellowing	Ok	
Safe Sept	Yes	Henry Schein	Pure	196 a	Tracks on surface	Nok	15.02.13
				196 b	Strong yellowing	Nok	
Pro Pad	Yes	DKL GmbH	Pure	197 a	Light tracks on surface	Ok	15.02.13
				197 b	Yellowing	Nok	

a: Stamskin et b: Batyline

REMARK: In any case, take a look at the disinfectant instruction notice before using it. For a long time using period, deterioration of padding material is not excluded. In that case, please contact us.